




MIC Test Strip

for antimicrobial susceptibility testing

Quantitative assay for determining the Minimum Inhibitory Concentration (M.I.C.)

MIC Test Strip is a quantitative assay for determining the Minimum Inhibitory Concentration (M.I.C.) of antimicrobial agents against microorganisms and for detecting the resistance mechanisms. **MIC Test Strip** are paper strips with special features * that are impregnated with a predefined concentration gradient of antibiotic, across 15 two-fold dilutions of a conventional M.I.C. method. On one side of the strip is indicated a M.I.C. scale in $\mu\text{g/mL}$ and a code that identify the antimicrobial agent. For ESBL, MBL, GRD, AmpC and KPC detection, the double-sided gradient carries the appropriate diagnostic reagents. **MIC Test Strip** are available in a large variety of configurations. Each configuration is available in packages of 10, 30 and 100 tests.

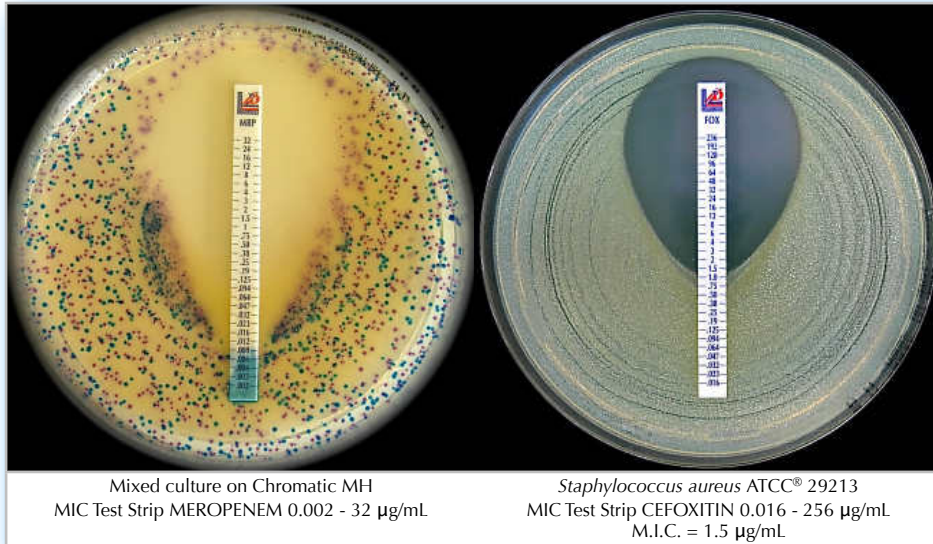


Mixed culture on Chromatic MH
MIC Test Strip MEROPENEM 0.002 - 32 $\mu\text{g/mL}$

Staphylococcus aureus ATCC® 29213
MIC Test Strip CEFOXITIN 0.016 - 256 $\mu\text{g/mL}$
M.I.C. = 1.5 $\mu\text{g/mL}$

method principle

When the **MIC Test Strip** is applied onto an inoculated agar surface, the preformed exponential gradient of antimicrobial agent is immediately transferred to the agar matrix. After 18 hours incubation or longer, a symmetrical inhibition ellipse centered along the strip is formed. The MIC is read directly from the scale in terms of $\mu\text{g/mL}$ at the point where the edge of the inhibition ellipse intersects the strip **MIC Test Strip**. Other growth/inhibition patterns may also be seen for resistance detection methods.

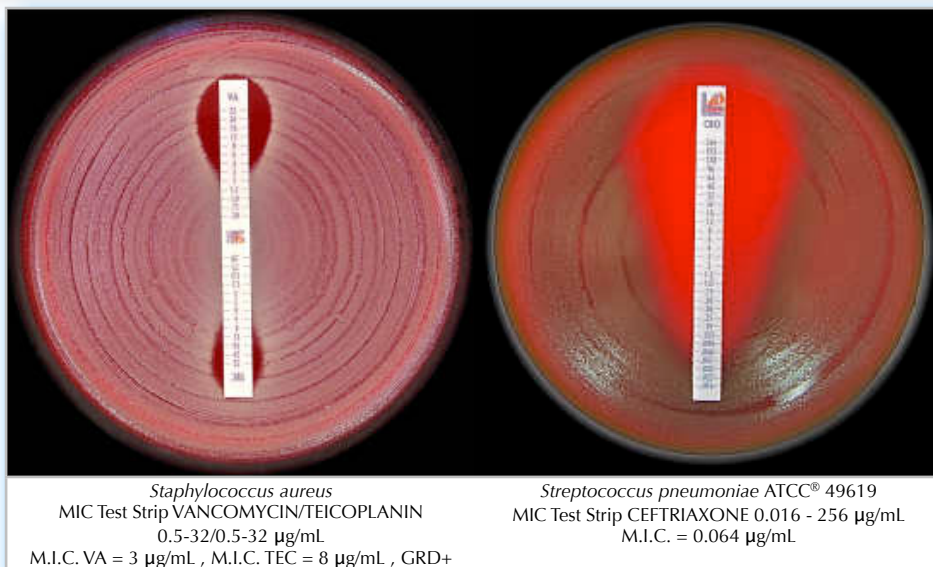


Staphylococcus aureus
MIC Test Strip VANCOMYCIN/TEICoplanin
0.5-32/0.5-32 $\mu\text{g/mL}$
M.I.C. VA = 3 $\mu\text{g/mL}$, M.I.C. TEC = 8 $\mu\text{g/mL}$, GRD+

Streptococcus pneumoniae ATCC® 49619
MIC Test Strip CEFTRIAXONE 0.016 - 256 $\mu\text{g/mL}$
M.I.C. = 0.064 $\mu\text{g/mL}$

BIBLIOGRAPHY

- CLSI M100-S22 , 2012. Performance Standards for Antimicrobial Susceptibility Testing.
- CLSI M7-A9, 2012. Methods for Dilution Antimicrobial Susceptibility Tests for Bacteria that Grow Aerobically.
- CLSI M11-A7, 2007. Methods for Dilution Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M11-S1 Performance Standards for Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M27-A3. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Approved Standard - Third Edition.
- CLSI M27-S3. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Third Informational supplement.
- EUCAST. Breakpoint tables for interpretation of MICs and zone diameters Version 2.0 , January 2012.

* MIC Test Strip Patent Pending

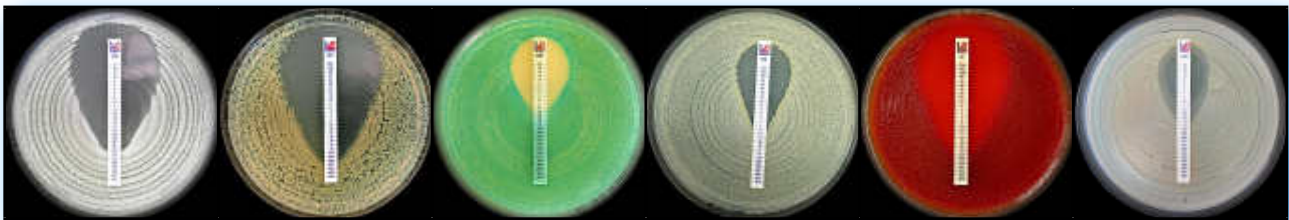

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		$\mu\text{g/mL}$	Code	Packaging	Ref.
MIC Test Strip	AMIKACIN	0.016 - 256	AK	10 strips 30 strips 100 strips	920181 92018 920180
MIC Test Strip	AMOXICILLIN	0.016 - 256	AML	10 strips 30 strips 100 strips	920211 92021 920210
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2/1) <i>CLSI recommended</i>	0.016 - 256*	AUG	10 strips 30 strips 100 strips	920241 92024 920240
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2 $\mu\text{g/mL}$) <i>EUCAST recommended</i>	0.016 - 256*	AMC	10 strips 30 strips 100 strips	921801 92180 921800
MIC Test Strip	AMPICILLIN	0.016 - 256	AMP	10 strips 30 strips 100 strips	920031 92003 920030
MIC Test Strip	AMPICILLIN*-SULBACTAM (2/1) <i>CLSI recommended</i>	0.016 - 256*	AMS	10 strips 30 strips 100 strips	920271 92027 920270
MIC Test Strip	AMPICILLIN*-SULBACTAM (4 $\mu\text{g/mL}$) <i>EUCAST recommended</i>	0.016 - 256*	SAM	10 strips 30 strips 100 strips	921811 92181 921810
MIC Test Strip	AZITHROMYCIN	0.016 - 256	AZM	10 strips 30 strips 100 strips	920301 92030 920300
MIC Test Strip	AZTREONAM	0.016 - 256	ATM	10 strips 30 strips 100 strips	920331 92033 920330
MIC Test Strip	BACITRACIN	0.016 - 256	BA	10 strips 30 strips 100 strips	920191 92019 920190
MIC Test Strip	CEFACLOR	0.016 - 256	CEC	10 strips 30 strips 100 strips	920361 92036 920360
MIC Test Strip	CEFEPIME	0.016 - 256	FEP	10 strips 30 strips 100 strips	921261 92126 921260
MIC Test Strip	CEFIXIME	0.016 - 256	CFM	10 strips 30 strips 100 strips	920601 92060 920600
MIC Test Strip	CEFOPERAZONE*-SULBACTAM (2/1)	0.016 - 256*	CPS	10 strips 30 strips 100 strips	920231 92023 920230
MIC Test Strip	CEFOTAXIME	0.002 - 32	CTX	10 strips 30 strips 100 strips	920071 92007 920070
MIC Test Strip	CEFOTAXIME	0.016 - 256	CTX	10 strips 30 strips 100 strips	920061 92006 920060
MIC Test Strip	CEFOTETAN	0.016 - 256	CTT	10 strips 30 strips 100 strips	920201 92020 920200
MIC Test Strip	CEFOXITIN	0.016 - 256	FOX	10 strips 30 strips 100 strips	920661 92066 920660
MIC Test Strip	CEFPIROME	0.016 - 256	CR	10 strips 30 strips 100 strips	920081 92008 920080
MIC Test Strip	CEFPODOXIME	0.016 - 256	PX	10 strips 30 strips 100 strips	920051 92005 920050
MIC Test Strip	CEFTAROLINE	0.016 - 256	CPT	10 strips 30 strips 100 strips	920491 92049 920490
MIC Test Strip	CEFTAZIDIME	0.016 - 256	CAZ	10 strips 30 strips 100 strips	921381 92138 921380
MIC Test Strip	CEFTIZOXIME	0.002 - 32	CZX	10 strips 30 strips 100 strips	920161 92016 920160
MIC Test Strip	CEFTRIAXONE	0.016 - 256	CRO	10 strips 30 strips 100 strips	920421 92042 920420
MIC Test Strip	CEFTRIAXONE	0.002 - 32	CRO	10 strips 30 strips 100 strips	920431 92043 920430
MIC Test Strip	CEFUROXIME	0.016 - 256	CXM	10 strips 30 strips 100 strips	921291 92129 921290

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEPHALOTHIN	0.016 - 256	KF	10 strips 30 strips 100 strips	920391 92039 920390
MIC Test Strip	CHLORAMPHENICOL	0.016 - 256	C	10 strips 30 strips 100 strips	920751 92075 920750
MIC Test Strip	CIPROFLOXACIN	0.002 - 32	CIP	10 strips 30 strips 100 strips	920451 92045 920450
MIC Test Strip	CLARITHROMYCIN	0.016 - 256	CLR	10 strips 30 strips 100 strips	920481 92048 920480
MIC Test Strip	CLINDAMYCIN	0.016 - 256	CD	10 strips 30 strips 100 strips	920721 92072 920720
MIC Test Strip	CLOXACILLIN	0.016 - 256	CX	10 strips 30 strips 100 strips	920441 92044 920440
MIC Test Strip	COLISTIN	0.016 - 256	CS	10 strips 30 strips 100 strips	921411 92141 921410
MIC Test Strip	COLISTIN	0.064 - 1024	CS	10 strips 30 strips 100 strips	921421 92142 921420
MIC Test Strip	DAPTOMYCIN (Includes Ca ²⁺)	0.016 - 256	DAP	10 strips 30 strips 100 strips	921451 92145 921450
MIC Test Strip	DORIPENEM	0.002 - 32	DOR	10 strips 30 strips 100 strips	920401 92040 920400
MIC Test Strip	DOXYCYCLINE	0.016 - 256	DXT	10 strips 30 strips 100 strips	921561 92156 921560
MIC Test Strip	ENROFLOXACIN	0.002 - 32	ENR	10 strips 30 strips 100 strips	920131 92013 920130
MIC Test Strip	ERTAPENEM	0.002 - 32	ETP	10 strips 30 strips 100 strips	921571 92157 921570
MIC Test Strip	ERYTHROMYCIN	0.016 - 256	E	10 strips 30 strips 100 strips	920511 92051 920510
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	0.016 - 256	FOS	10 strips 30 strips 100 strips	920781 92078 920780
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	0.064 - 1024	FOS	10 strips 30 strips 100 strips	920791 92079 920790
MIC Test Strip	FUSIDIC ACID	0.016 - 256	FU	10 strips 30 strips 100 strips	920021 92002 920020
MIC Test Strip	GATIFLOXACIN	0.002 - 32	GAT	10 strips 30 strips 100 strips	920111 92011 920110
MIC Test Strip	GEMIFLOXACIN	0.002 - 32	GEM	10 strips 30 strips 100 strips	920351 92035 920350
MIC Test Strip	GENTAMICIN	0.016 - 256	CN	10 strips 30 strips 100 strips	920091 92009 920090
MIC Test Strip	GENTAMICIN	0.064 - 1024	CN	10 strips 30 strips 100 strips	920101 92010 920100
MIC Test Strip	IMIPENEM	0.002 - 32	IMI	10 strips 30 strips 100 strips	920541 92054 920540
MIC Test Strip	KANAMYCIN	0.016 - 256	K	10 strips 30 strips 100 strips	920341 92034 920340
MIC Test Strip	LEVOFLOXACIN	0.002 - 32	LEV	10 strips 30 strips 100 strips	920811 92081 920810
MIC Test Strip	LINEZOLID	0.016 - 256	LNZ	10 strips 30 strips 100 strips	921351 92135 921350
MIC Test Strip	MECILLINAM	0.016 - 256	MEC	10 strips 30 strips 100 strips	920171 92017 920170
MIC Test Strip	MEROPENEM	0.002 - 32	MRP	10 strips 30 strips 100 strips	920841 92084 920840
MIC Test Strip	METRONIDAZOLE	0.016 - 256	LZ	10 strips 30 strips 100 strips	920871 92087 920870
MIC Test Strip	MOXIFLOXACIN	0.002 - 32	MXF	10 strips 30 strips 100 strips	920901 92090 920900
MIC Test Strip	MUPIROCIN	0.064 - 1024	MUP	10 strips 30 strips 100 strips	920381 92038 920380

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	MINOCYCLINE	0.016 - 256	MN	10 strips 30 strips 100 strips	920321 92032 920320
MIC Test Strip	NALIDIXIC ACID	0.016 - 256	NA	10 strips 30 strips 100 strips	921321 92132 921320
MIC Test Strip	NETILMICIN	0.016 - 256	NET	10 strips 30 strips 100 strips	920931 92093 920930
MIC Test Strip	NORFLOXACIN	0.016 - 256	NOR	10 strips 30 strips 100 strips	920961 92096 920960
MIC Test Strip	NITROFURANTOIN	0.032 - 512	F	10 strips 30 strips 100 strips	920221 92022 920220
MIC Test Strip	OFLOXACIN	0.002 - 32	OFX	10 strips 30 strips 100 strips	920991 92099 920990
MIC Test Strip	OXACILLIN	0.016 - 256	OX	10 strips 30 strips 100 strips	920151 92015 920150
MIC Test Strip	PENICILLIN G	0.002 - 32	P	10 strips 30 strips 100 strips	921031 92103 921030
MIC Test Strip	PENICILLIN G	0.016 - 256	P	10 strips 30 strips 100 strips	921021 92102 921020
MIC Test Strip	PIPERACILLIN	0.016 - 256	PIP	10 strips 30 strips 100 strips	921051 92105 921050
MIC Test Strip	PIPERACILLIN*-TAZOBACTAM (4 µg/mL)	0.016 - 256*	TZP	10 strips 30 strips 100 strips	921081 92108 921080
MIC Test Strip	POLYMYXIN B	0.064 - 1024	PB	10 strips 30 strips 100 strips	920041 92004 920040
MIC Test Strip	QUINUPRISTIN-DALFOPRISTIN	0.002 - 32	QDA	10 strips 30 strips 100 strips	920261 92026 920260
MIC Test Strip	RIFAMPICIN	0.002 - 32	RD	10 strips 30 strips 100 strips	920011 92001 920010
MIC Test Strip	RIFAMPICIN	0.016 - 256	RD	10 strips 30 strips 100 strips	920251 92025 920250
MIC Test Strip	SPECTINOMYCIN	0.064 - 1024	SPC	10 strips 30 strips 100 strips	920141 92014 920140
MIC Test Strip	SPIRAMYCIN	0.002 - 32	SP	10 strips 30 strips 100 strips	920461 92046 920460
MIC Test Strip	STREPTOMYCIN	0.064 - 1024	S	10 strips 30 strips 100 strips	921111 92111 921110
MIC Test Strip	SULBACTAM	0.016 - 256	SUL	10 strips 30 strips 100 strips	920281 92028 920280
MIC Test Strip	SULFAMETHOXAZOLE	0.064 - 1024	SMX	10 strips 30 strips 100 strips	920311 92031 920310
MIC Test Strip	TEICOPLANIN	0.016 - 256	TEC	10 strips 30 strips 100 strips	920121 92012 920120
MIC Test Strip	TELAVANCIN	0.002 - 32	TLV	10 strips 30 strips 100 strips	920521 92052 920520
MIC Test Strip	TELAVANCIN	0.016 - 256	TLV	10 strips 30 strips 100 strips	920531 92053 920530
MIC Test Strip	TEMOCILLIN	0.064 - 1024	TMO	10 strips 30 strips 100 strips	920291 92029 920290
MIC Test Strip	TETRACYCLINE	0.016 - 256	TE	10 strips 30 strips 100 strips	921141 92114 921140
MIC Test Strip	TICARCILLIN*-CLAVULANIC ACID (2 µg/mL)	0.016 - 256*	TTC	10 strips 30 strips 100 strips	921171 92117 921170
MIC Test Strip	TIGECYCLINE	0.016 - 256	TGC	10 strips 30 strips 100 strips	921441 92144 921440
MIC Test Strip	TOBRAMYCIN	0.016 - 256	TOB	10 strips 30 strips 100 strips	921211 92121 921210
MIC Test Strip	TOBRAMYCIN	0.064 - 1024	TOB	10 strips 30 strips 100 strips	921201 92120 921200
MIC Test Strip	TRIMETHOPRIM	0.002 - 32	TM	10 strips 30 strips 100 strips	920371 92037 920370

MIC Test Strip for ANTIMICROBIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	TRIMETHOPRIM*-SULFAMETHOXAZOLE (1/19)	0.002 - 32*	SXT	10 strips 30 strips 100 strips	921231 92123 921230
MIC Test Strip	VANCOMYCIN	0.016 - 256	VA	10 strips 30 strips 100 strips	920571 92057 920570

MIC Test Strip for ANTIFUNGAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	AMPHOTERICIN B	0.002 - 32	AMB	10 strips 30 strips 100 strips	921531 92153 921530
MIC Test Strip	ANIDULAFUNGIN	0.002 - 32	AND	10 strips 30 strips 100 strips	921551 92155 921550
MIC Test Strip	CASPOFUNGIN	0.002 - 32	CAS	10 strips 30 strips 100 strips	921541 92154 921540
MIC Test Strip	FLUCONAZOLE	0.016 - 256	FLU	10 strips 30 strips 100 strips	921471 92147 921470
MIC Test Strip	FLUCYTOSINE	0.002 - 32	FC	10 strips 30 strips 100 strips	921491 92149 921490
MIC Test Strip	ITRACONAZOLE	0.002 - 32	ITC	10 strips 30 strips 100 strips	921481 92148 921480
MIC Test Strip	KETOCONAZOLE	0.002 - 32	KE	10 strips 30 strips 100 strips	921511 92151 921510
MIC Test Strip	MICAFUNGIN	0.002 - 32	MYC	10 strips 30 strips 100 strips	921821 92182 921820
MIC Test Strip	POSACONAZOLE	0.002 - 32	POS	10 strips 30 strips 100 strips	921521 92152 921520
MIC Test Strip	VORICONAZOLE	0.002 - 32	VO	10 strips 30 strips 100 strips	921501 92150 921500

MIC Test Strip for ANTIMYCOBACTERIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	ETHAMBUTOL	0.016 - 256	EB	10 strips 30 strips 100 strips	921701 92170 921700
MIC Test Strip	ETHIONAMIDE	0.016 - 256	ET	10 strips 30 strips 100 strips	921721 92172 921720
MIC Test Strip	ISONIAZIDE	0.016 - 256	IZ	10 strips 30 strips 100 strips	921711 92171 921710


MIC Test Strip ESBL for confirmation of Extended Spectrum Beta-Lactamase					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFEPIME/CEFEPIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.064-4	FEP/FEL	10 strips 30 strips 100 strips	921611 92161 921610
MIC Test Strip	CEFOTAXIME/CEFOTAXIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.016-1	CTX/CTL	10 strips 30 strips 100 strips	921601 92160 921600
MIC Test Strip	CEFTAZIDIME/CEFTAZIDIME+CLAVULANIC ACID (4 µg/mL)	0.5-32 / 0.064-4	CAZ/CAL	10 strips 30 strips 100 strips	921591 92159 921590

MIC Test Strip MBL for detection of Metallo Beta-Lactamase					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	4-256 / 1-64	IMI/IMD	10 strips 30 strips 100 strips	921621 92162 921620
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	0.125-8 / 0.032-2	IMI/IMD	10 strips 30 strips 100 strips	921661 92166 921660
MIC Test Strip	MEROPENEM / MEROPENEM + EDTA	0.125-8 / 0.032-2	MRP/MRD	10 strips 30 strips 100 strips	921651 92165 921650


MIC Test Strip for KPC detection					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	MEROPENEM / MEROPENEM + BORONIC ACID	0.125-8 / 0.032-2	MRP/MBO	10 strips 30 strips 100 strips	921671 92167 921670
MIC Test Strip	ERTAPENEM / ERTAPENEM + BORONIC ACID	0.125-8 / 0.032-2	ETP/EBO	10 strips 30 strips 100 strips	921681 92168 921680

MIC Test Strip GRD for detection of Glycopeptide Resistance					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	VANCOMYCIN / TEICoplanin	0.5-32 / 0.5-32	VA/TEC	10 strips 30 strips 100 strips	921631 92163 921630

MIC Test Strip for AmpC detection					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFOTETAN / CEFOTETAN+CLOXACILLIN	0.5-32 / 0.5-32	CTT/CXT	10 strips 30 strips 100 strips	921641 92164 921640

Accessories			
Description		Packaging	Ref.
Automatic Rotation Plater 90 mm plates 14-110 rpm		1 unit	96795
Automatic Rotation Plater PRO (14-210 rpm)		1 unit	96796
Turntable extra wide (140 mm plates)		1 unit	96797
Wireless Foot Pedal		1 unit	96801
Steel Foot Pedal		1 unit	96798
Plastic Foot Pedal		1 unit	96799
STORAGE TUBE WITH SILICA GEL		10 tubes	96719
0.5 McFarland		1 tube	80400
1 McFarland		1 tube	80401
2 McFarland		1 tube	80402

Ready to use culture media			
DESCRIPTION	FORMAT	PACKAGING	REF.
MUELLER HINTON AGAR for antimicrobial susceptibility testing of non-fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10031 10231
MUELLER HINTON AGAR + SHEEP BLOOD 5% for antimicrobial susceptibility testing of fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10131 11231
MUELLER HINTON FASTIDIOUS AGAR (Horse blood 5% + 20 mg/L β-NAD) EUCAST recommended: for antimicrobial susceptibility testing of <i>Streptococcus</i> spp., <i>Haemophilus</i> spp., and some other fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10132 11132
MUELLER HINTON AGAR + 2% NaCl for antimicrobial susceptibility testing of staphylococci with Oxacillin.	90 mm	20 plates	11206
CHROMATIC MH Chromogenic medium for the preliminary identification and susceptibility testing of bacteria directly from clinical and environmental specimens.	90 mm	20 plates	11618
BRUCELLA BLOOD AGAR with HEMIN AND VITAMIN K1 for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	10245
SCHADLER K AGAR (SHEEP BLOOD 5%) for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	11065
HAEMOPHILUS TEST AGAR for <i>Haemophilus</i> spp. susceptibility testing.	90 mm	20 plates	10080
MIDDLEBROOK 7H11 AGAR for antimicrobial susceptibility testing of mycobacteria.	90 mm	20 plates	10416
R.P.M.I. AGAR (2% glucose + MOPS) for antifungal susceptibility testing.	90 mm 140 mm	20 plates 10 plates	11509 10233

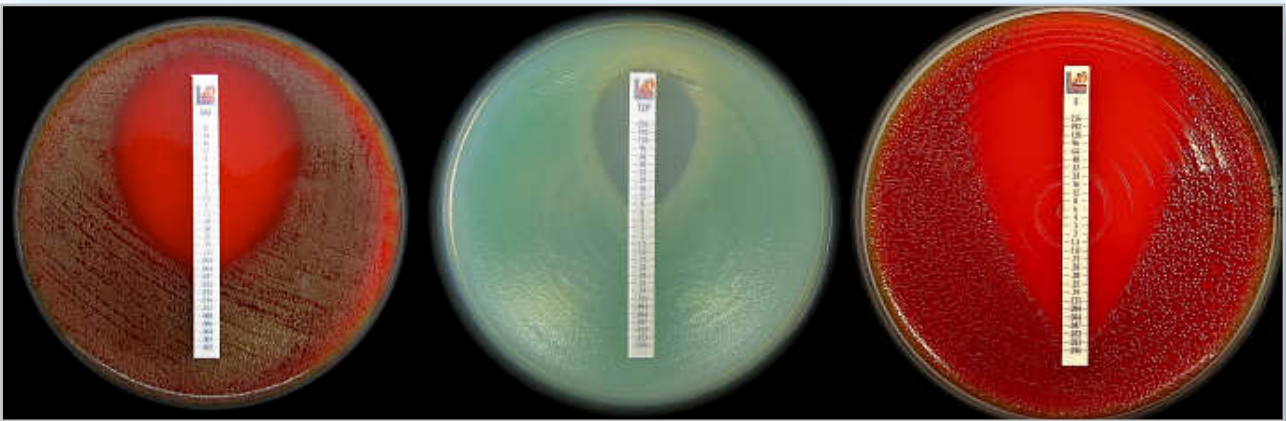

S. pneumoniae ATCC® 49619
MIC Test Strip IMI 0.002-32 µg/mL
M.I.C. = 0.094 µg/mL

P. aeruginosa ATCC® 27853
MIC Test Strip TZP 0.016-256 µg/mL
M.I.C. = 8 µg/mL


S. pneumoniae ATCC® 49619
MIC Test Strip E 0.016-256 µg/mL
M.I.C. = 0.032 µg/mL



E. coli ATCC® 25922
MIC Test Strip SXT 0.002-32 µg/mL
M.I.C. = 0.094 µg/mL

B. fragilis ATCC® 25285
MIC Test Strip AUG 0.016-256 µg/mL
M.I.C. = 0.25 µg/mL

E. coli ATCC® 25922
MIC Test Strip AMP 0.016-256 µg/mL
M.I.C. = 8 µg/mL

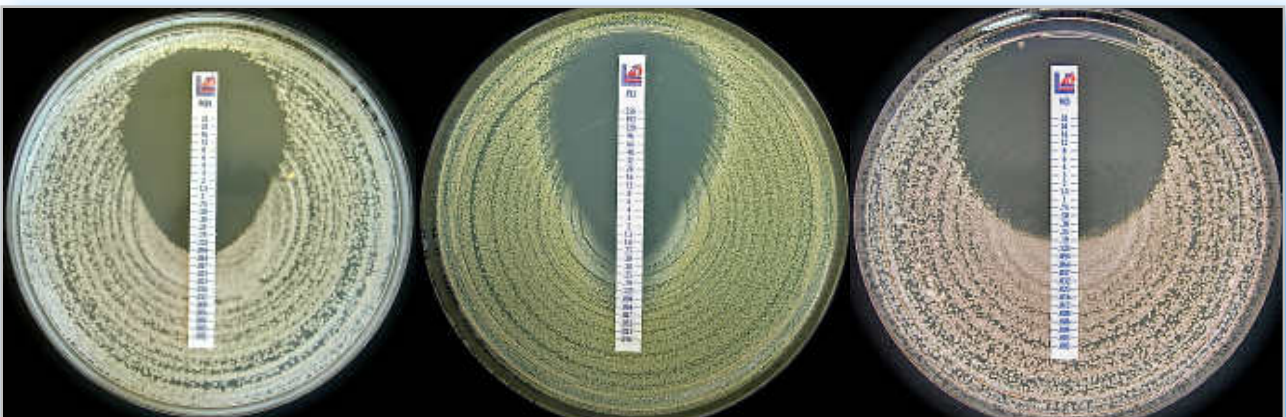

C. krusei ATCC® 6258
MIC Test Strip POS 0.002-32 µg/mL
M.I.C. = 0.094 µg/mL

S. aureus ATCC® 29213
MIC Test Strip FU 0.016-256 µg/mL
M.I.C. = 0.50 µg/mL

C. krusei
MIC Test Strip VO 0.002-32 µg/mL
M.I.C. = 0.25 µg/mL
















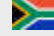















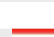
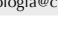
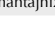
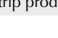
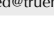




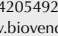
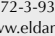

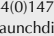


S. maltophilia ATCC® 13636
MIC Test Strip IMI/IMD 4-256/1-64 µg/mL
MBL+

E. coli DSM 22311
MIC Test Strip CTX/CTL 0.25-16/0.016-1 µg/mL
ESBL+

E. coli ATCC® 25922
MIC Test Strip CAZ/CAL 0.5-32/0.064-4 µg/mL
ESBL-

Contact your local dealer or Liofilchem headquarter for MIC Test Strip availability.

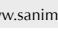 Australia Integrated Sciences Pty Limited Tel +61 2 9417 7866 Fax +61 2 9417 5066 www.integratedsci.com.au	 Egypt Media Medix Trade Tel +20 22598417 Fax +20 24519256 mediamo_co@yahoo.com	 Japan Alere Medical Co., Ltd. Tel +81353267393 Fax +81353267177 www.alere.co.jp	 Serbia Uni-Chem Tel: +381113282986 Fax: +381113281205 www.uni-chem.rs
 Austria Alere GmbH Tel +43(0)732771077 Fax +43(0)732771077 2391 www.alere.at	 Estonia Borsa Eesti OÜ Tel +3725050555 Fax +3726556158 borsa@borsa.ee	 Lebanon Biochemistry Marketing & Services sarl Tel +961 (9)912047 Fax +961 (9)912047 jechemal@inco.com.lb	 Slovakia Bio G spol. s r.o. Tel +421245526381 Fax +421245526382 www.biog.sk
 Benelux ELITech Benelux B.V.B.A Tel +32 (0)92820531 Fax +32 (0)92820532 c.notte@elitechgroup.com	 Finland Immuno Diagnostic Oy Tel +3583615370 Fax +35836822039 www.immunodiagnostic.fi	 Lithuania Biometrija Tel +37 052481824 fax +37 052400085 www.biometrija.lt	 Slovenia Mediline d.o.o. Tel +38618308050 Fax +38618308070 www.mediline.si
 Bulgaria Elta 90M Tel +35929839649 Fax +35929832211 www.elta90.com	 France i2a Tel +33467504800 Fax +33467170906 www.i2a.info	 New Zealand Point View Laboratories Ltd Tel +64 9 533 3380 Fax +64 9 533 3340 info@pointviewlaboratories.com	 South Africa Davies Diagnostics (PTY Ltd) Tel +27117777600 Fax +27117876917 www.daviesdiagnostics.co.za
 Canada Alere Canada Tel: (613) 271-1144 Fax: (613) 271-1148 www.alere.ca	 Germany bestbion dx GmbH Tel +49 2234 987 95 0 Fax +49 2234 987 95 29 bernd.stammel@bestbion.com	 Norway Montebello Diagnostics AS Tel 22 14 14 90 Fax 22 14 44 90 www.montebello.no	 Spain IZASA Distribuciones Técnicas, S.A. Tel: 902203090 Fax 902223366 www.izasa.es
 Costa Rica Inter-Biolab S.A. Tel: (506) 2226-0131 / (506) 2226-4322 Fax: (506) 2226-0136 ecoto@inter-biolab.com	 Greece Varelas S.A. Tel +302105281920 Fax +302105220926 microbiology@varelas.gr	 Pakistan Imran Scientific Co Tel 021-32210083 Fax 021-32210630 www.imransco.com	 Sweden BioNordika AB Tel 08 30 60 10 Fax 08 30 60 15 www.bionordika.se
 Croatia Tamiko Instruments Tel +38514683046 Fax +38514684811 www.tamiko.hr	 Hungary Frank Diagnosztika Kft. Tel 388-3114, 367-2430 Fax 368-5721 www.frank-diagn.hu	 Paraguay GT Scientific S.A. Tel 595 21 200134 Fax 595 21 221339 ventas@gtscientific.com.py	 Switzerland Alere GmbH Tel: +41 (0) 44 782 60 70 Fax: +41 (0) 44 782 60 77 www.aleregmbh.ch
 Cuba CPM Cuba Tel 2041432-2041433 Fax 2040987 microbiologia@cpm.co.cu	 Iran Saman Tajhiz Azma Tel +982188247651-2 Fax +982188278480 info@samantajhizazma.org	 Portugal Izasa Portugal Distribuicoes Tecnicas Lda. Tel +351 214161288 Fax +351 214176484 MIC Test Strip product line only	 Thailand Envimed Co., Ltd. Tel 66-2-434-0391 Fax 66-2-435-5962 envimed@truemail.co.th
 Cyprus Sundial Co. Ltd tel +357-25828828 fax +357-25828830 maria@sundial-cy.com	 Ireland Launch Diagnostics Ireland Tel +3530719151122 Fax +3530719152277 desruddy@launchdiagnostics.ie	 Romania SC. Sanimed International Impex SRL Tel 021.420.54.94 Fax 021.420.54.93 www.sanimed.ro	 Turkey Savas Medikal TIC. LTD. STI. Tel +90 2126357689 Fax: +90 2125238071 www.savasmedikal.com
 Czech Republic BioVendor Tel +420549124123 Fax +420549211465 www.biovendor.cz	 Israel ELDAN Electronic Instruments Co Ltd. Tel 972-3-9371116 Fax 972-3-9371121 www.eldan.biz	 Russia Agros-International, Zao Tel +7 (812) 303-8234 Fax +7 (812) 303-8236 agros@peterlink.ru	 United Kingdom Launch Diagnostics Ltd Tel +44(0)1474874426 Fax +44(0)1474872388 enquiries@launchdiagnostics.com
 Denmark Simoco Diagnostics ApS Tel +4545939614 Fax +4545889611 www.simoco.dk	 Italy Liofilchem Tel +39 0858930745 Fax +39 0858930330 www.liofilchem.it	 Saudi Arabia Gamsco Tel +966-38125606 Fax +966-38125607 www.gamscomed.com	 Uruguay Eubiosis S.R.L. Tel +59824863020 Fax +59824865084 www.eubiosis.com



LIOFILCHEM® s.r.l.

Via Scozia zona ind.le, 64026 Roseto degli Abruzzi (Te) Italy

Tel. +39 0858930745 Fax +39 0858930330 www.liofilchem.net liofilchem@lioilchem.net



ref. 6553003
Rev.7 / 17.07.2012